

FUNCTIONAL SPLINT LIGHT/CHILD

HIGHLIGHTS AND INDICATIONS Adult Light #10015

SIZES: Length 237mm

SUPPORTS: Forearm, Wrist, Palm

DESCRIPTION: WRIST HAND ORTHOSIS, thermoplastic transparent splint. Light and durable customizable functional splint. Brings wrist to optimal functional position while enabling holding other tools. Static but slightly flexible splint which allows rest to wrist and free movement of fingers. Provides rest to wrist with minimal support. Keeps wrist in dorsiflex position that allows using the fingers and enables use of more potential strength of fingers. Thermoplastic transparent splint. Can be worn overnight and/or during the day


INDICATIONS: : Sprains, Carpal Tunnel Syndrome, rheumatoid arthritis, reduction of wrist motion, reduction of pain, immobilization of wrist to prevent flexion

CONTRAINDICATIONS: This splint does not immobilize the distal radioulnar joint
Not for acute phase of most wrist fractures
Do not apply directly on open wound

CUSTOMIZATION HIGHLIGHTS

- Choose splint direction according to large arch that follows thumb line.
- Make sure to bend top edge of splint below proximal transverse skin crease of palm to allow maximal comfort inside palm.
- Radial line (line at base of thumb - trapezium bone) must be below base of thumb to allow free movement of thumb.
- Top splint part must be below proximal transverse skin crease of palm to allow free finger movement.

STEPS

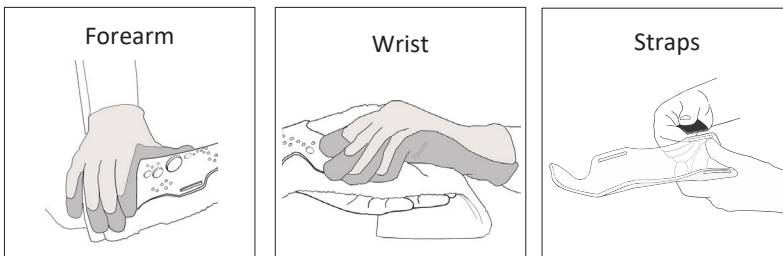

HIGHLIGHTS

