

FUNCTIONAL THUMB SPLINT | HIGHLIGHTS AND INDICATIONS

Adult #10049

SIZES: Adult length 265mm

SUPPORTS: Forearm, Wrist, Palm, Thumb, Base of Thumb.

DESCRIPTION: STATIC WRIST HAND THUMB ORTHOSIS . Wrist and thumb splint which reduces thumb motion and immobilizes the wrist. Allows free movement of the fingers. Provides stabilization and protection of both the thumb and the wrist. Brings the wrist to optimal functional position while fixing the thumb. Static thermoplastic transparent splint.


INDICATIONS CMC arthritis, de Quatrain's tenosynovitis, Treatment of sprain of the thumb or wrist, reduction of inflammation, prevention of wrist ulnar deviation and thumb flexion, reduction of pain, reduction of inflammation.

CONTRAINDICATIONS: Do not apply in acute phase. Not for the acute phase of most wrist Fractures. Do not apply directly on open wound.

CUSTOMIZATION HIGHLIGHTS

- Make sure the bent splint parts do not touch each other
- The top part of the splint must be below the proximal transverse skin crease of the palm to allow free finger movement
- Make sure to bend the top edge of the splint below the proximal transverse skin crease of the palm to allow maximal comfort inside the palm
- Check this splint with the straps and only then repair
- The thumb should be fixed at the base and free at the top

STEPS

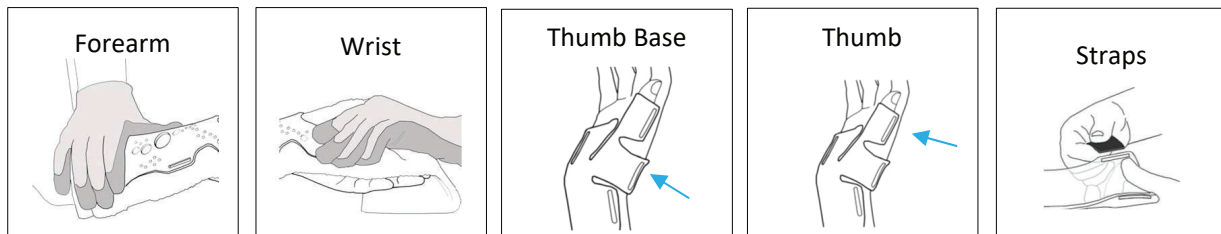

HIGHLIGHTS

