


FORK HOLDER

KIT Adult #10068, Child #10066

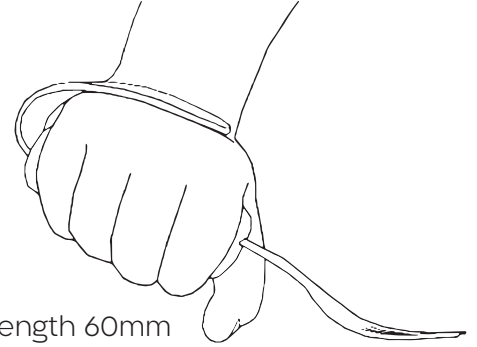

Sponge: Adult Diameter 32, length 100mm, Child Diameter 24, Length 60mm

Remark: Provided without a fork


Description: Sponge tube and customizable plastic handle to add to fork or any other tool suitable. Plastic handle bent according to palm shape and assists stabilize fork in hand and prevents tool from falling. Enables using fork independently for eating. Can be adjusted to other tools.

Recommendation: People with difficulty using their fingers to grab handle, finger manipulation impairments, low fine motor skills, tremor. Worn over palm or fingers but never on MP joints.


Customization Highlights

- Bend handle gently from middle without applying high pressure
- Wear under or above MP joint, but never on it
- Curve top part of handle to fit palm
- Adjust grasping angle with fork for maximal comfort
- Insert narrow area of handle in one side of sponge and fork from other side of sponge
- Lastly, hand should come inside holder almost freely, without falling from hand.

STEPS

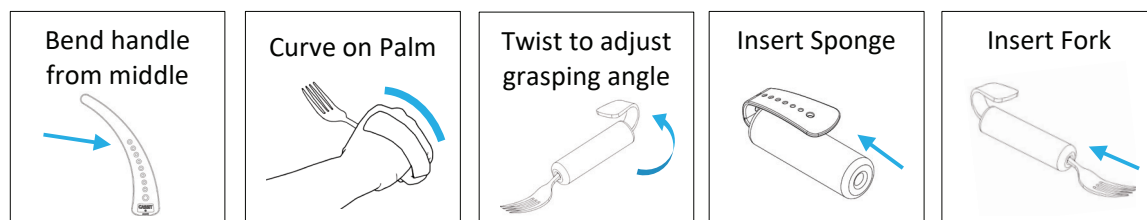

HIGHLIGHTS

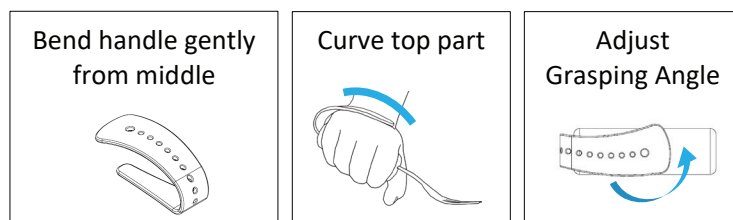

CASSIT™